

Hladni Val Face Shield

Protect yourself and others.

STANDARD NUMBER

Hladni Val Face Shield satisfies the requirements of the European Directive for Personal Protection Equipment (PPE) 2016/425 and has been manufactured in accordance with the requirements of EN166:2001.

CLASSIFICATION - FACE SHIELD

Hladni Val Face Shield meets the requirement to protect against splashes of liquids, however it does not protect from dust or impacts.

STORAGE AND MAINTENANCE

To maintain optimal condition store your Hladni Val Face Shield at room temperature in a case or bag. Similarly, when carrying or transporting your protective Hladni Val Face Shield, keep it in a casing and do not expose it to other items, such as tools or sharp objects, where damage could occur.

Regularly clean with warm soapy water and rinse thoroughly. Dry with a lint free cloth (e.g. microfiber) or tissue to avoid scratching.

PRE-USE CHECKS

To ensure your on-going safety, check all parts are not loose and tighten if necessary. Please be aware that damage or deterioration can occur during storage and/or transit. Should there be signs of damage or scratches, please dispose of your Hladni Val Face Shield and replace immediately.

ADDITIONAL INFORMATION

Item Number: M.O-Z.M-0001

Manufacturer: Hladni Val d.o.o.

Address: Il. Industrijski Odvojak, Industrijski Park, 35400, Nova Gradiška, Hrvatska

EU Declaration of Conformity:

<https://hladnival.hr/hladni-val-face-shield-0001-eu-declaration-of-conformity-en.pdf>

PRODUCT CODE MARKINGS

The lens and frame are marked in accordance with EN166:2001. Please refer to the tables below for an explanation of these markings:

FIELDS OF USE	
Liquid droplets or splashes	3
Large dust particles > 5 microns	4
Gas and fine dust particles < 5 microns	5
Electrical short circuit arc	8
Molten metal and hot solids	9
MECHANICAL STRENGTH	
Extra strong	S
Low energy impact	F
Medium energy impact	B
High energy impact	A
OPTICAL PERFORMANCE	
Continuous work	1
Intermittent work	2
Occasional work, but must not be worn continuously.	3
Resistance to surface damage by fine particles (optional)	K
Resistance to fogging (optional)	N




PLEASE NOTE

If the lens and frame do not carry identical markings relating to strength, then the whole face shield is classified at the lower rating.


It is possible to wear prescription eyewear underneath your Hladni Val Face Shield. However, any collision encountered may impact on the prescription eyewear. As such, the protection stated on the eye protector may be reduced.

Please ensure the information above is followed, or your safety may be significantly reduced.




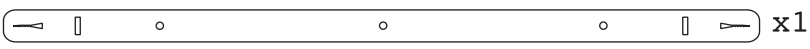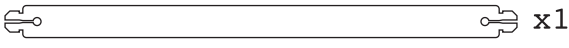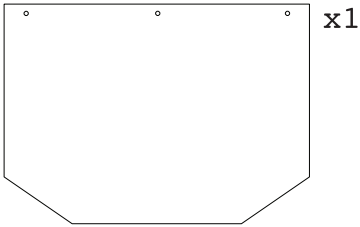

1

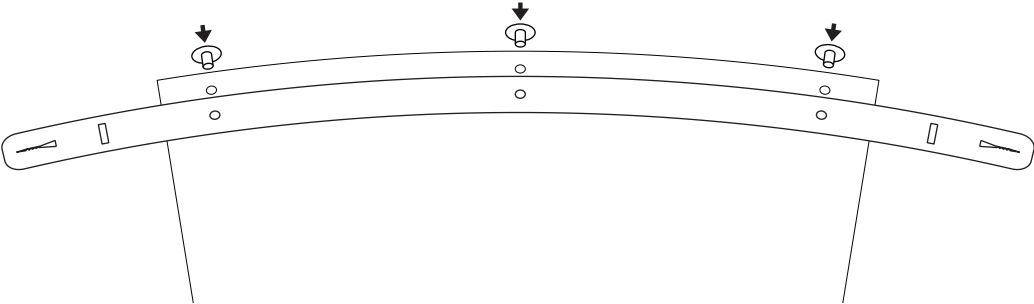

2

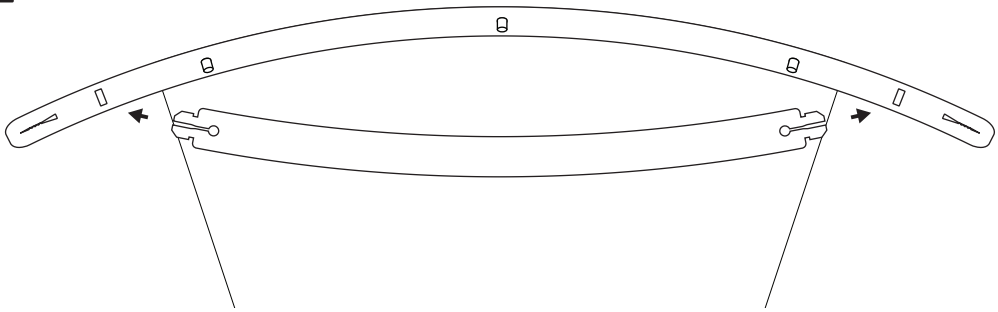

3

